



# Instrument Lubricant

Clean Start Chemicals

Specifically designed for Skytron washer-disinfector systems.



# Instrument Lubricant Cleaning and Disinfection Solution

Containing a corrosion inhibitor and lubricant derived from natural sources, Skytron's Instrument Lubricant is specially crafted to provide lubrication and safeguard surgical instruments, hinges, box locks, and general equipment throughout the autoclaving process.

## Recommended Uses:

- Medical/surgical instruments

## Advantages:

- Contains an anticorrosive agent to help prevent corrosion
- The gentle formula promotes ease of use
- Extends instrument life by leaving a protective coating
- Bacteriostatic

## Technical Data:

- pH 6.5 to 7.5
- Non-corrosive
- Milk bath dilution 1 part solution to 6 parts water
- Automated equipment 1/8 to 1/2 oz. per gallon of water

## Packaging:

- Four 1 gallon containers – CSTART-9801
- Two 2.5 gallon containers – CSTART-9825
- One 5 gallon container – CSTART-9805

Skytron provides safe and efficient healthcare solutions delivered through a local representative network that values long-term relationships and comprehensive service.

Focused on providing the highest quality solutions while delivering the lowest cost of ownership, we prioritize transparency and accountability in our actions.

**To learn more, visit our website at [www.skytron.com](http://www.skytron.com)**



5085 Corporate Exchange Blvd. SE  
Grand Rapids, MI 49512  
1.800.SKYTRON (759-8766)  
email: [info@skytron.com](mailto:info@skytron.com)  
[www.skytron.com](http://www.skytron.com)



REV1 12-23  
PN L2-010-242