

# Skytron Integration Quick User Guide

Skytron CAST Support (Image Capture and Record): **1-855-4-677477**




## Table of Contents


Video Integration Pages 1-6

CAST Pages 7-22


# Home Page - Integration

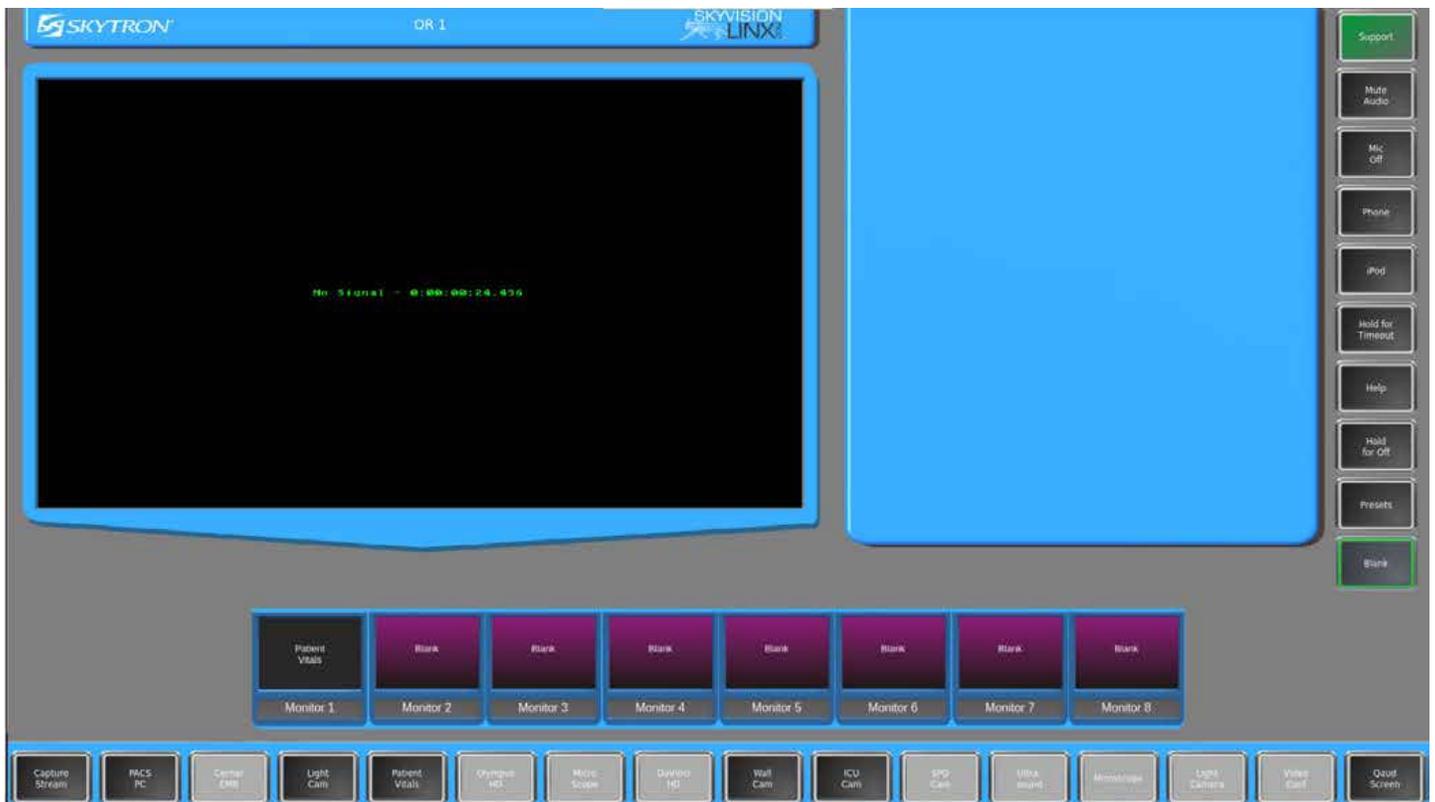

# For standard image routing, select any source

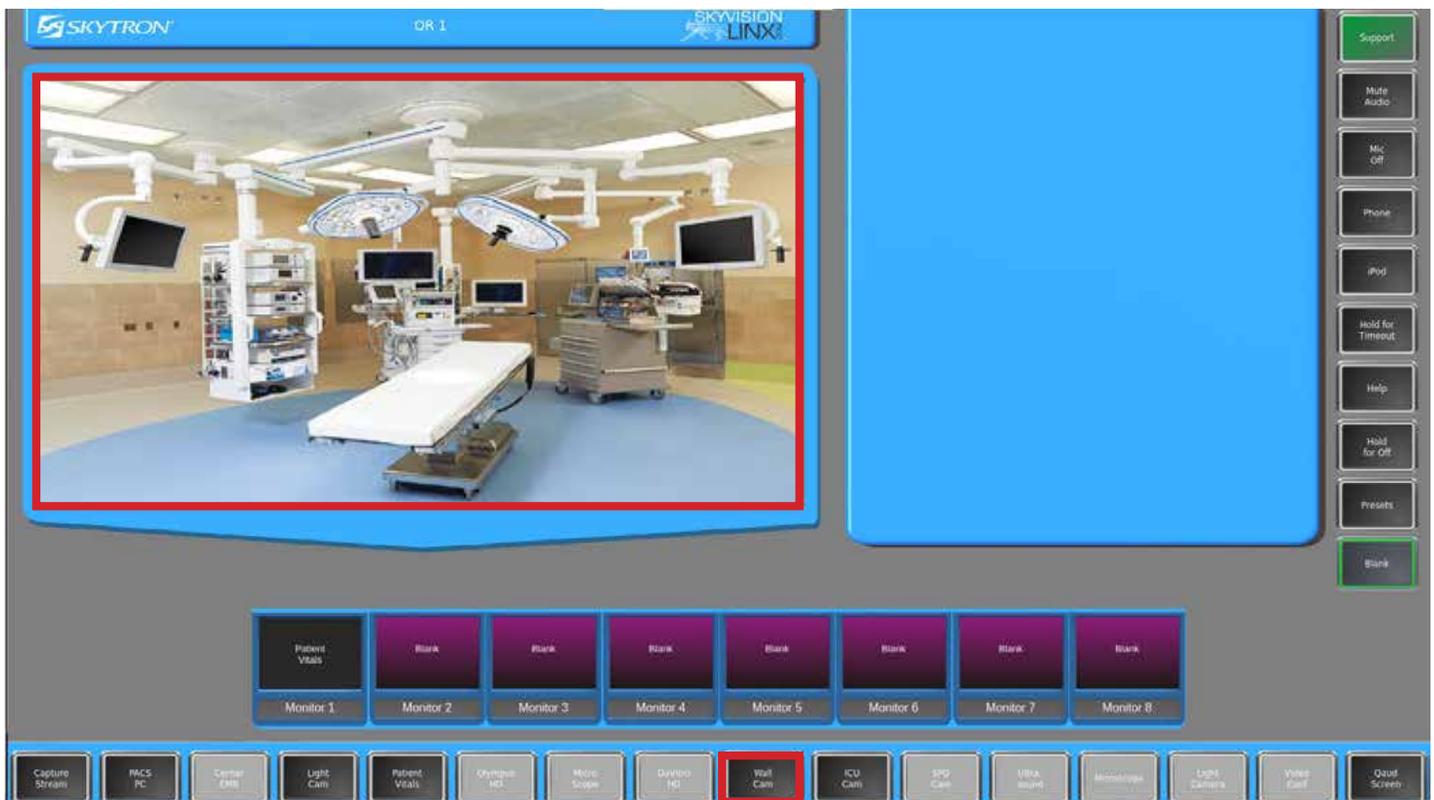

Select any monitor and your image will show on the corresponding monitor in the field

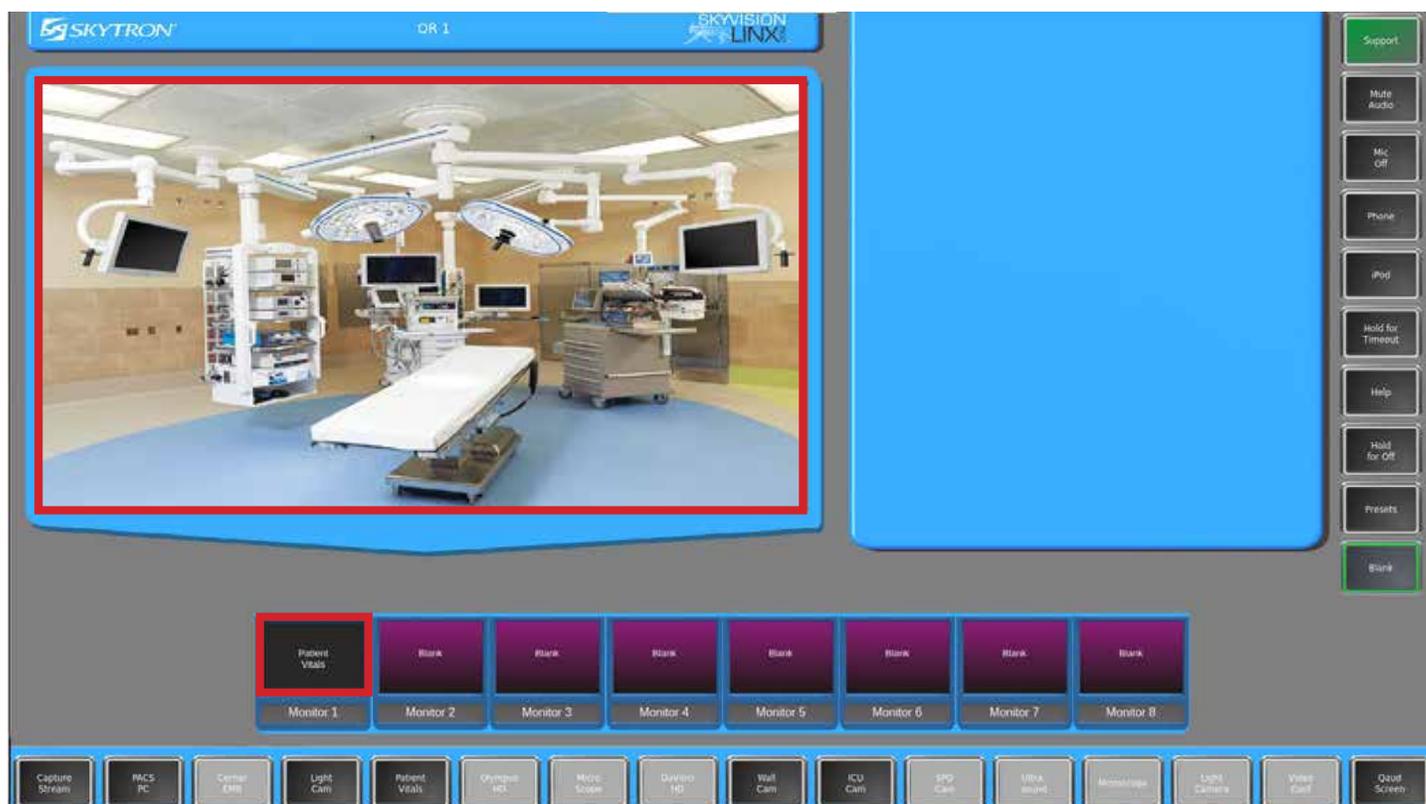

# To load a preset, select Preset



Press and hold the corresponding preset you would like to load until you see the yellow box\*



\*Dark gray: Active sources  
Light gray: Inactive sources

# Your preset is now loading and will automatically populate all monitors



# Capturing Pictures and Video (CAST)




# To capture images or record, select CAST PC



# Select capture sources



Select the corresponding active source you would like to capture and press close.\*

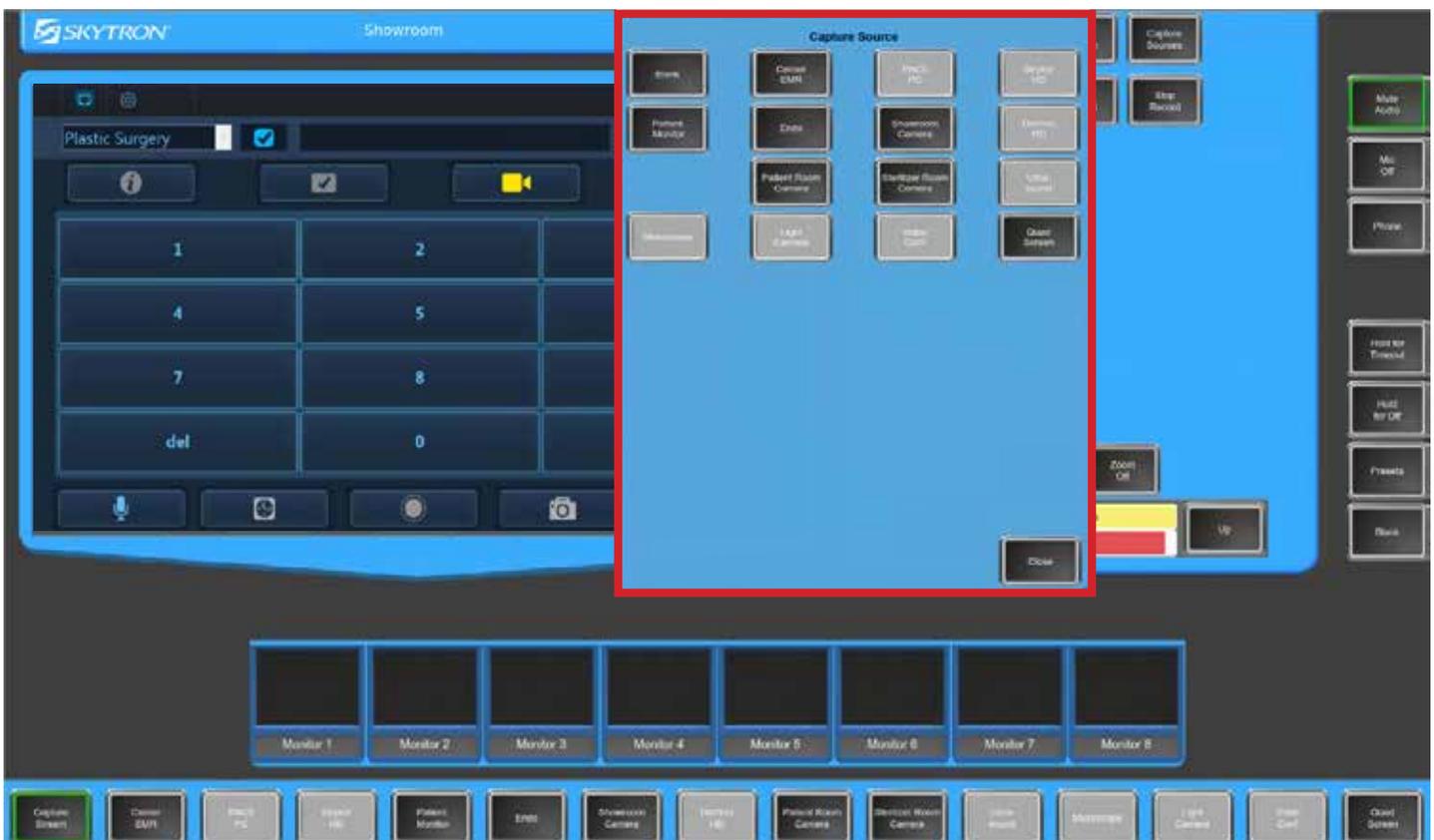

\*Dark gray: Active sources  
Light gray: Inactive sources


If an active source is selected, you will see a preview of the image



# Touch the screen to enter the full screen CAST application



# Add a patient by selecting the patient icon and type in information

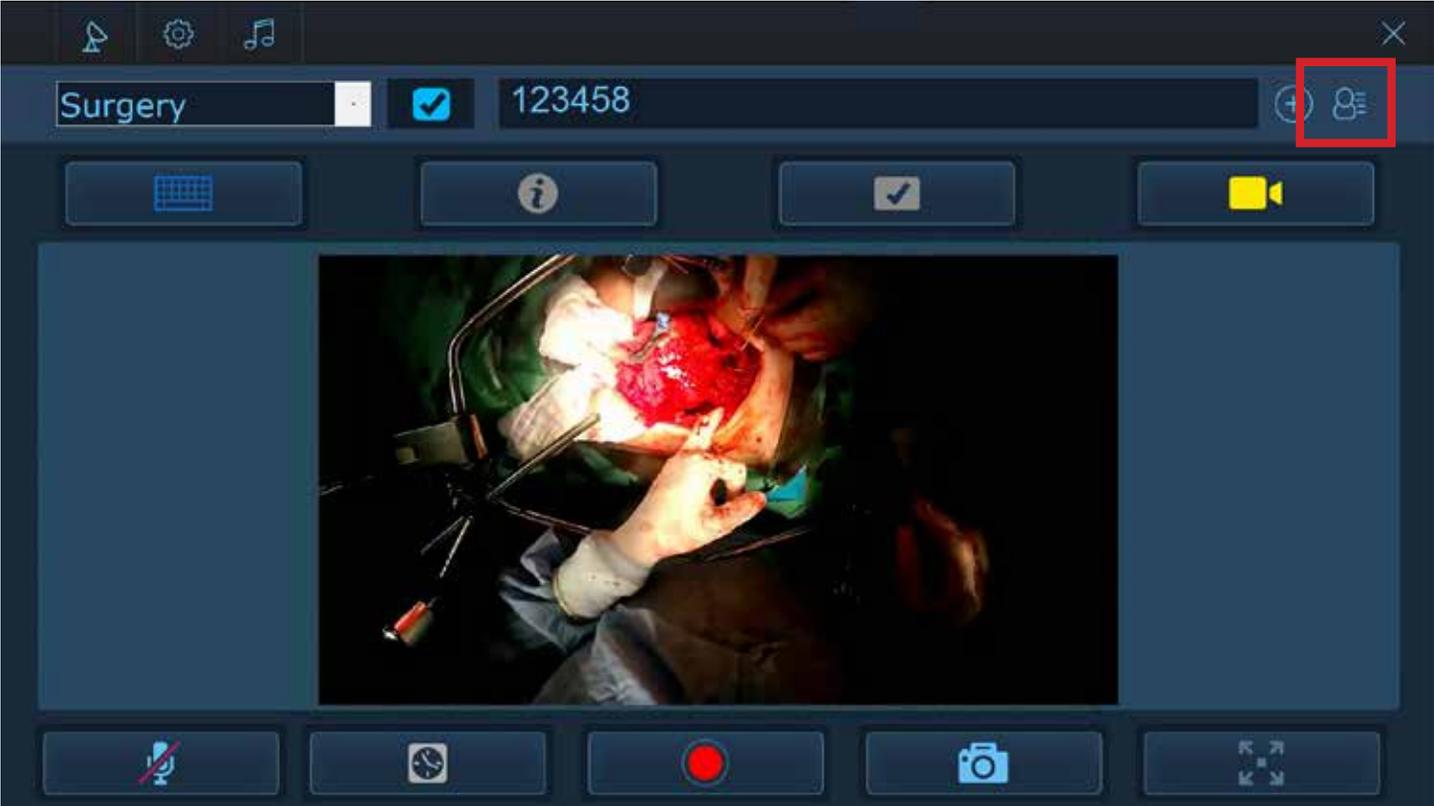


Once you have successfully opened a case, your source will appear along with all patient information

The screenshot displays a mobile application interface with a dark blue theme. A central form titled "Patient Identification" is overlaid on a background screen. The form includes the following fields and controls:

- Gender:** A dropdown menu currently set to "Male".
- First Name:** A text input field.
- Last Name:** A text input field.
- DOB:** Three separate dropdown menus for "Month" (set to "January"), "Day" (set to "1"), and "Year" (set to "1980").
- Address:** A text input field.
- Keypad:** A numeric keypad with buttons for digits 1-9, 0, a "del" button, and a "> abc" button.
- Navigation:** "OK" and "Cancel" buttons at the bottom of the form.

The background interface shows a "Surgery" header, an information icon, and various navigation and utility icons. The top right corner of the app is labeled "ORPHEUS X".


From here you can take a picture

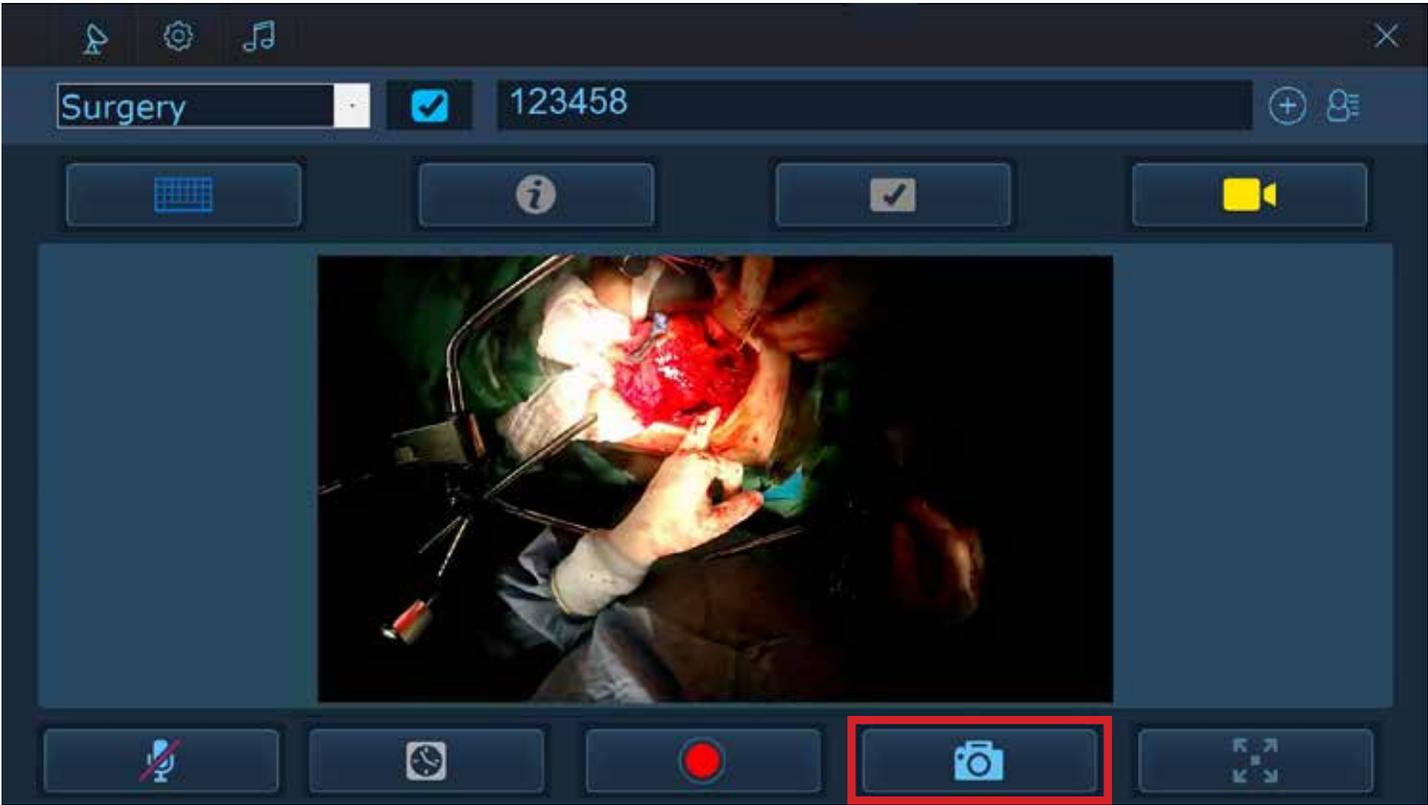

All pictures taken are shown on the right



Select the center red circle to record a video

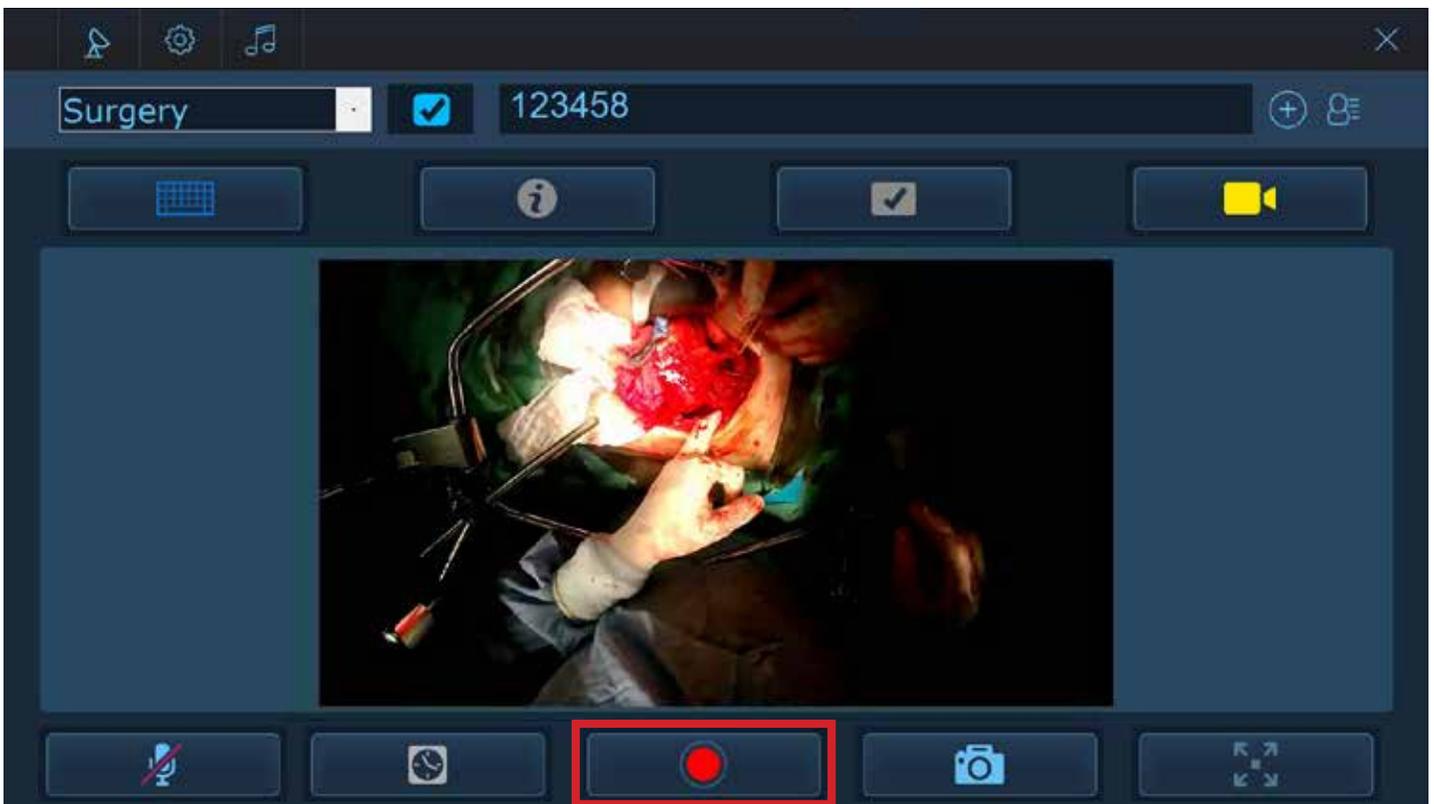


You are able to take additional pictures even while recording.


Press the pause button to create a video clip. Press play again to resume the recording




**When you are certain the case is complete and no additional pictures/video is needed, select the square box to end procedure**

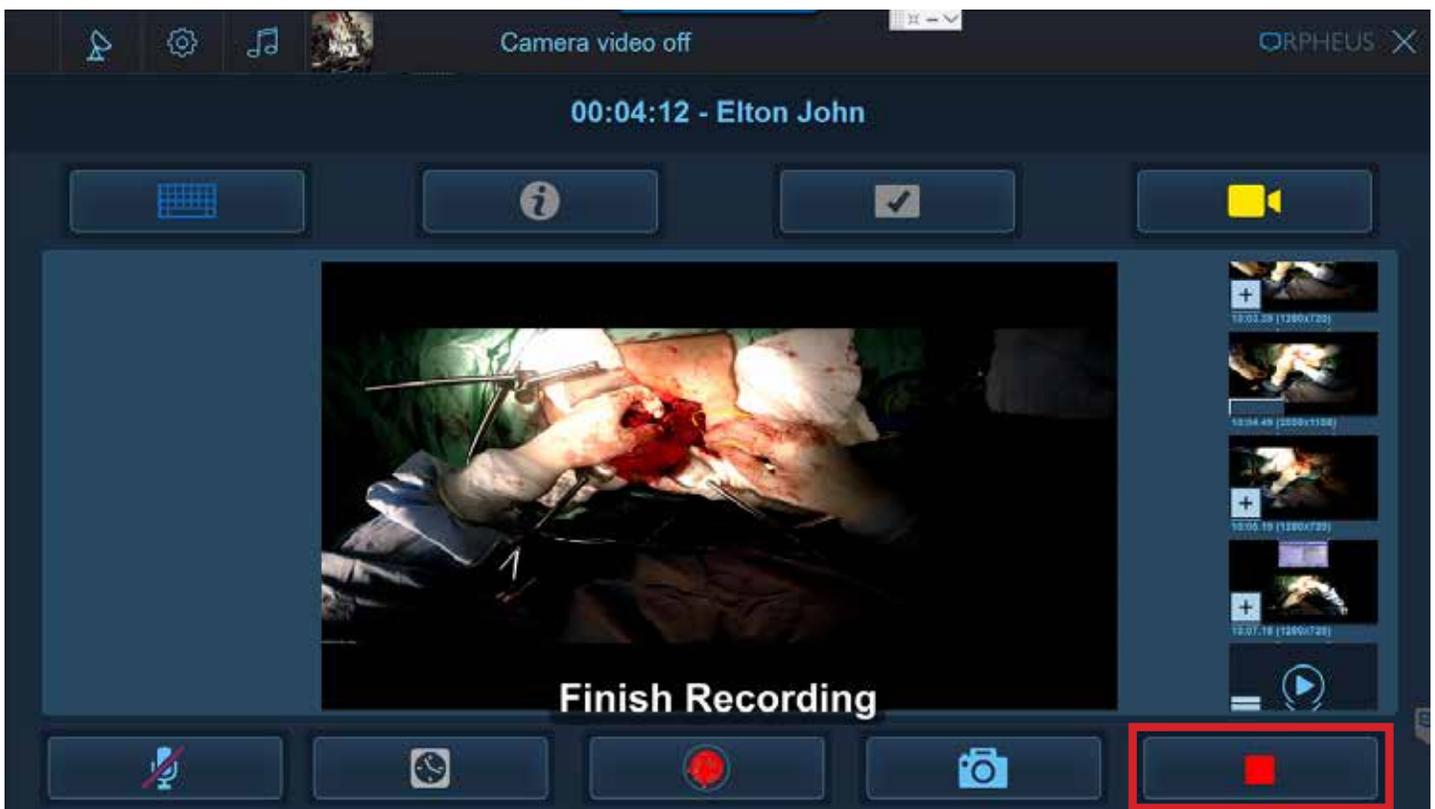

Once you end the case, it will take you back to the main CAST screen

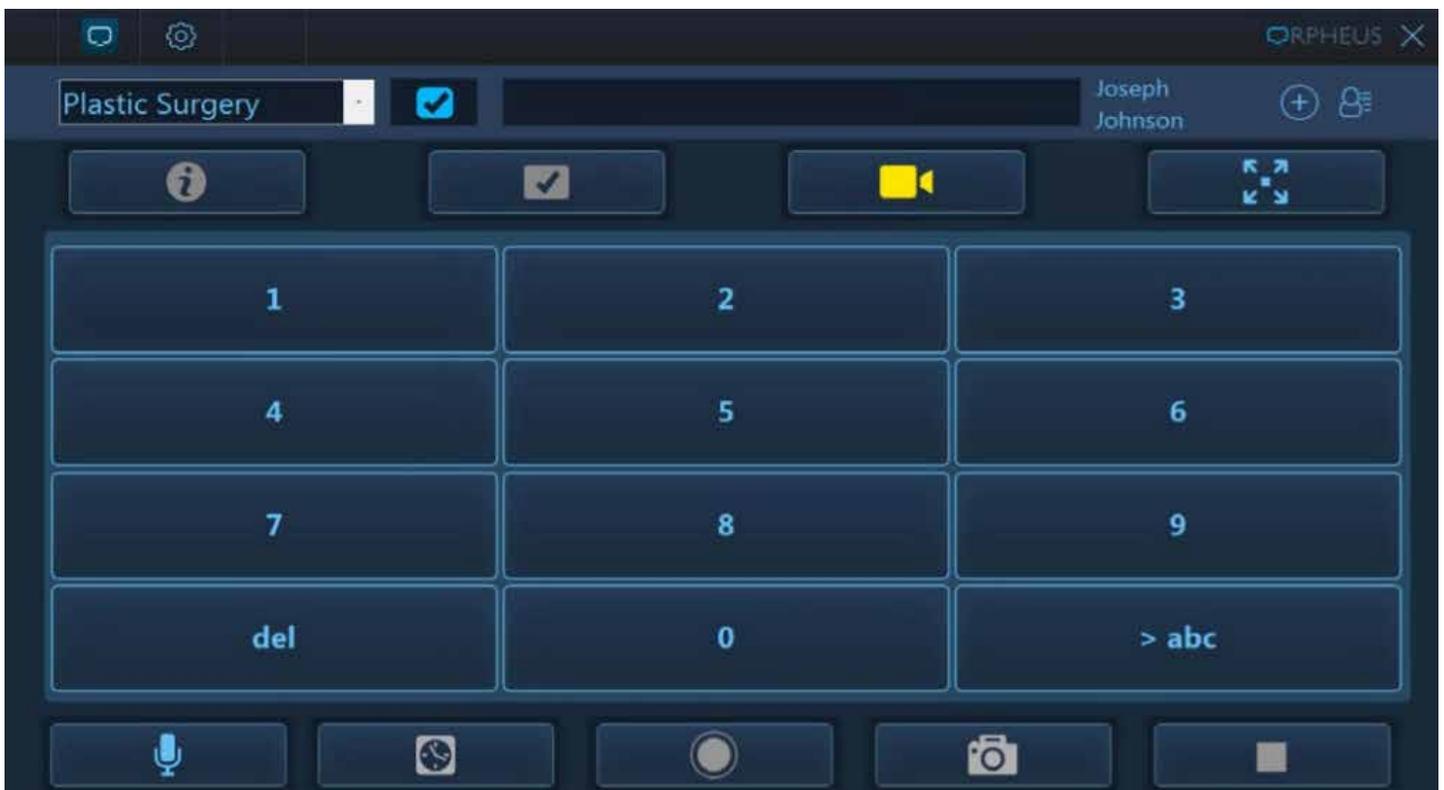

Click the “X” shown to get back to the Integration homepage

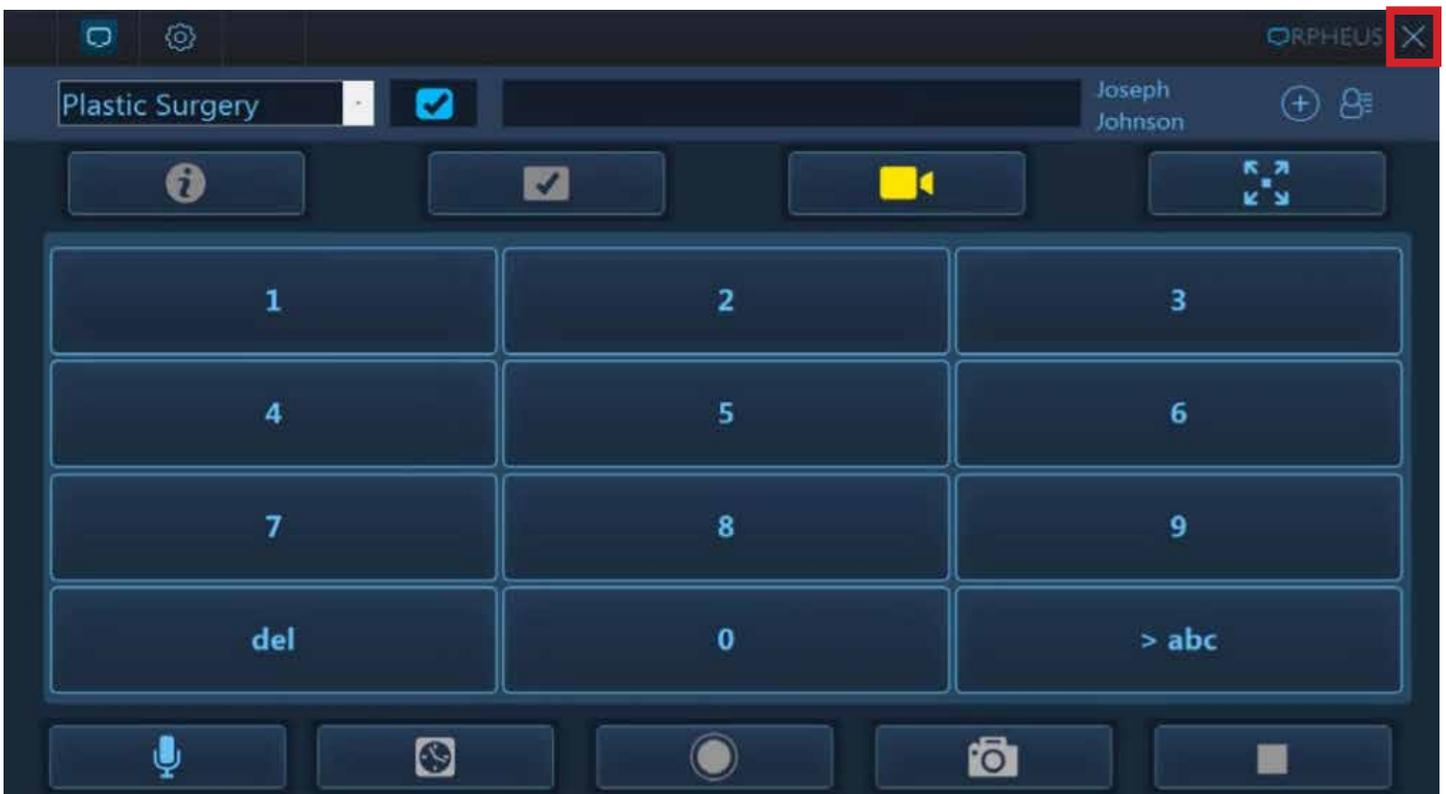

# Integration homepage

