



# Skytron

## Surgical Table Pads

### Great Fluid Protection and Durability

Unlike standard pads which allow liquids to enter between stitches and into the foam layers, Skytron pads have welded seams that are tightly sealed for cleanliness and durability. Ultrasonic welders are used to form a leak-proof barrier, 'welding' the seams and preventing liquids from entering the inner foam. Cover material offers four-way stretch to reduce damage from shear force.



Welded Seam Pad



Standard Stitched Pad


### Optimal Comfort and Equalized Pressure

Skytron table pads are composed of three layers of foam, providing optimal comfort and equalized pressure regardless of patient size or procedure length. Soft, body forming, layers are reinforced with different density foams to provide lasting support.

1" viscoelastic memory foam to equalize pressure

1/2" high density foam to provide long lasting support

1/2" vinyl nitril closed cell foam to strengthen the softer layers above

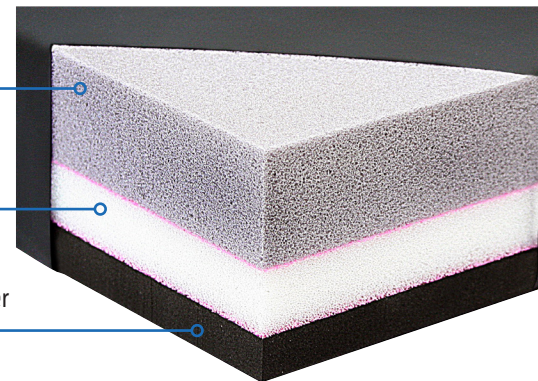

### Air Vent

Air enters and leaves the pad via a covered vent, allowing the foam to expand and contract with the patient while keeping fluids out.




# Skytron


## Surgical Table Pads

### Find the Right Pad for Your Table

Part numbers for Skytron pads and pad sets follow a consistent format. The numbering system always starts at the head end of the table and moves towards the foot end. In case of side-by-side pads, such as the split leg attachment, the first number will always be on the patient's left. Part numbers for pad sets will only have seven characters, while part numbers for specific pads will have eight.

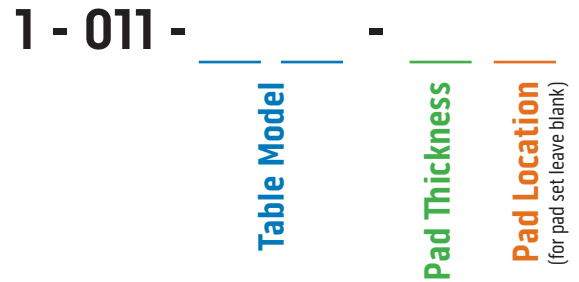

### Pad Location

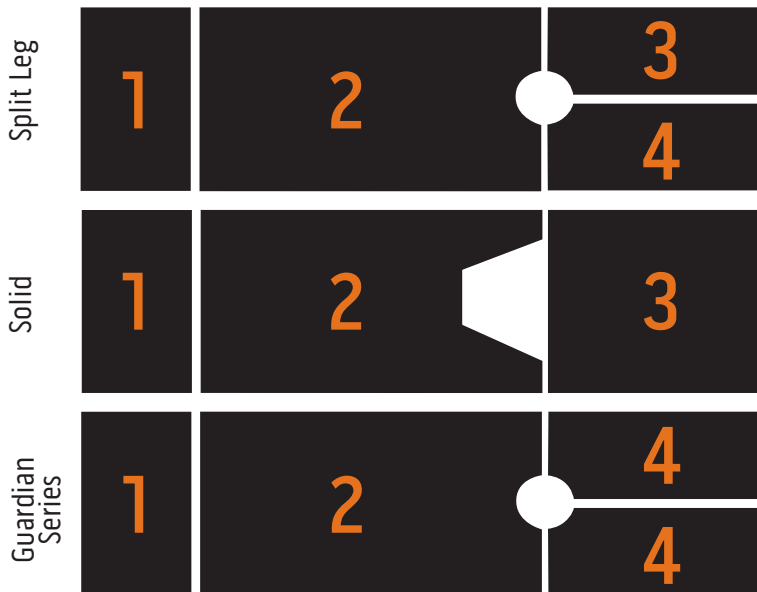

### Example

- 1 - 011 - 03 - 2    3003 table, 2" thick, pad set
- 1 - 011 - 05 - 43    3500B table, 4" thick, leg section
- 1 - 011 - 01 - 24    6000 table, 2" thick, split leg, patient's right
- 1 - 011 - 06 - 24    Guardian Series table, 2" thick, split leg, both legs

### Table Model

|    |            |   |
|----|------------|---|
| 01 | Split Leg  | 1200, 5000, 6000  |
| 02 | Single Leg | 300, 1201(K), 5001, 6001(K), 6002, 6002B, 6500, 6500HD, 6600 6600B          |
| 03 | Single Leg | 3000, 3001, 3002, 3003, 3008  |
| 04 | Single Leg | 3100  |
| 05 | Single Leg | 3500, 3500B   |
| 06 | Single Leg | 3501B, 3501C, 3502, 3600 series, Guardian series (Split leg sold as a set)  |
| 07 | Single Leg | 1700  |
| 08 | Single Leg | 6700 Series   |
| 09 | Single Leg | 5601E   |
| 11 | Single Leg | 6300 series   |
| 12 | Single Leg | 3501B, 3501C, 3502 and 3600 series bariatric pad set with 5" side extension |
| 13 | Single Leg | 6700 series bariatric pad set, with 5" side extension                       |
| 15 | Single Leg | 1602  |
| 16 | Split Leg  | 1602-SL   |

Skytron provides safe and efficient healthcare solutions delivered through a local representative network that values long-term relationships and comprehensive service.

Focused on providing the highest quality solutions while delivering the lowest cost of ownership, we prioritize transparency and accountability in our actions.

To learn more, visit our website at [www.skytron.com](http://www.skytron.com)



5085 Corporate Exchange Blvd. SE  
 Grand Rapids, MI 49512  
 1.800.SKYTRON (759-8766)  
 email: [info@skytron.com](mailto:info@skytron.com)  
[www.skytron.com](http://www.skytron.com)



REV7 03-24  
 PN L2-010-39